# Honigum-Heavy

## Inhalt | Contents

|                                      |     |
|--------------------------------------|-----|
| Gebrauchsinformation – Deutsch       | 4   |
| Instructions for use – English       | 14  |
| Mode d'emploi – Français             | 24  |
| Instrucciones de uso – Español       | 35  |
| Istruzioni per l'uso – Italiano      | 45  |
| Instruções de utilização – Português | 56  |
| Gebruiksaanwijzing – Nederlands      | 67  |
| Brugsanvisning – Dansk               | 77  |
| Bruksanvisning – Svenska             | 87  |
| Instrukcja użycia – Polski           | 97  |
| Информация по применению – Русский   | 108 |
| Kullanım kılavuzu – Türkçe           | 119 |

## Produktbeschreibung

---

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## Product Description

---

Honigum-Heavy is a precision pre-impression material based on A-silicones.

The material is available in the Automix safety cartridge and in the ready-to-use pre-filled MixStar cartridge, which is suitable for use in the DMG MixStar-eMotion and other commercially available mixing machines.

## Indications

---

- Crown and bridge impressions
- Inlay and onlay impressions
- All types of pickup impressions, e.g. for implants and telescopes

## Basic Safety Instructions

---

- For dental use only!
- Keep out of reach of children!


## Interactions with other materials

---


- Rinse residues of retraction solutions thoroughly with water, otherwise these in some instances can increase the setting time.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Never allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC- or PE-based gloves.
- Do not mix Honigum-Heavy with condensation curing silicone or polyether impression materials.
- Surpluses of material on methacrylate basis can change the setting properties of silicone impression materials. If a restoration has been made previously, clean the preparation and surrounding tissue carefully with gauze soaked in alcohol and water spray before taking the impression.

## Using the MixStar cartridge in the DMG MixStar-eMotion

- Follow the instructions for use for the “MixStar-eMotion”!

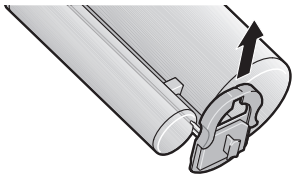
1. Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge.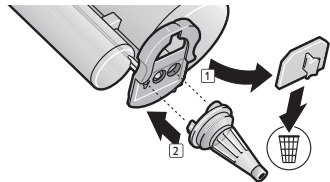


2. Remove the cap from the cartridge and discard. Attach a new mixing tip.

**Note:** In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are available as refill packs.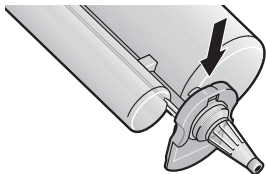

3. Secure the mixing tip by sliding the locking mechanism into the locked position.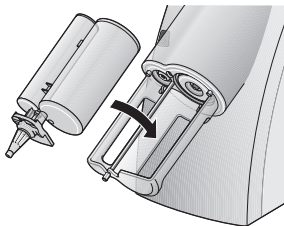

4. Insert the cartridge into the MixStar-eMotion. A brief acoustic signal confirms the correct insertion of the cartridge.

**Note:** Prepare fresh cartridges for use by extruding and discarding the first 3 cm of impression material.

**Note:** After the application, leave the used tip on the cartridge to cap unused material.

### Using the MixStar cartridge in mixing units from other manufacturers

- ▶ Follow the instructions of the mixing unit you are using!
- ▶ First insert the cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- ▶ In this case, proceed as described under “Changing the mixing tip” on page 17.

## Changing the mixing tip

- Follow the instructions for use for the chosen mixing unit!


**Note:** Attach a new mixing tip before each mixing process.

1. Slide the locking mechanism upwards.
2. Remove the old mixing tip and any hardened material blocking the cartridge openings.
3. Place the mixing tip securely on the respective cartridge opening.

**Note:** In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are available as refill packs.

4. Lock the mixing tip in place with the locking mechanism.

## Using the Automix cartridge

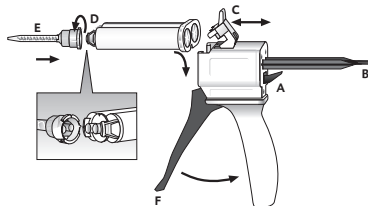### Insert cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

**Note:** Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

### Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Note:** In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are available as refill packs.

**Note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

### Applying the material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [G] of the Automix-Dispenser and it is then applied directly.

**Note:** After the application, leave the used mixing tip on the cartridge to cap unused material!

### Remove cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

### Instructions for use

---

- ▶ For the individual times refer to the “Technical data” on page 21.
- ▶ Honigum-Heavy can be used in all common impression trays.
- ▶ In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive for A-silicone should be applied (e.g. Tray Adhesive, DMG).

## Recommended Use

---

### Double-mix impressions

1. Fill the tray with Honigum-Heavy. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

**Note:** To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

- Syringe the preparation with Honigum-Light. Follow the associated instructions for use!

**Note:** The filling of the impression tray and syringing should be completed at the same time, because the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

2. Position the tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.

3. Remove the impression from the mouth once the setting time has ended; see “Removal” on page 20.

### Correction impression

**Note:** No more than 30 minutes are permitted to pass between the pre-impression and correction impression nor should alcohol be used for cleaning during this time, otherwise it is not possible to guarantee a firm bond between the two impression materials.

1. Cut generous drainage channels into the pre-impression for the wash and remove imperfections.
2. After repositioning, thoroughly rinse the impression with water and dry.
3. Apply Honigum-Light around the preparation and on the pre-impression. Follow the associated instructions for use!
4. Position the tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.

5. Remove the impression from the mouth once the setting time has ended; see “Removal” on page 20.

## Removal

---

1. Remove the impression parallel to the longitudinal axis of the prepared teeth.
2. Then proceed as described under “Cleaning and disinfection of the final impression” on page 20.

## Cleaning and disinfection of the final impression

---

- ▶ Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression materials.
- ▶ Follow the instructions of the disinfectant solution’s manufacturer precisely!

- ▶ Leave the final impression to dry and store until casting; see “Storage of the impression” on page 21.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; [www.dgzmk.de](http://www.dgzmk.de)
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; [www.ada.org](http://www.ada.org)

## Casting and galvanization

---

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873; Type 4, e.g. Fujirock II, GC). The impression can be copper-plated or silver-plated.

## Storage of the impression

- Store at room temperature (15–25 °C/ 59–77 °F) in a dry place!
- Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

## Technical data

| Honigum-Heavy                     |  |  |
|-----------------------------------|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|
| Working time* [min]               | ≤ 2:00                                                                            | ≤ 2:15                                                                            |
| Intraoral setting time [min]      | ≥ 3:15                                                                            | ≥ 3:15                                                                            |
| Dimensional change after 24 h [%] | ≤ 0.40                                                                            | ≤ 0.40                                                                            |
| Compression set [%]               | ≤ 0.50                                                                            | ≤ 0.50                                                                            |
| Strain in compression [%]         | ≈ 1.4                                                                             | ≈ 1.4                                                                             |
| Shore-A hardness                  | ≈ 67                                                                              | ≈ 67                                                                              |

| Honigum-Heavy                     | <br>Fast | <br>Fast |
|-----------------------------------|---------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------|
| Working time* [min]               | ≤ 1:15                                                                                      | ≤ 1:15                                                                                      |
| Intraoral setting time [min]      | ≥ 2:00                                                                                      | ≥ 2:00                                                                                      |
| Dimensional change after 24 h [%] | ≤ 0.40                                                                                      | ≤ 0.40                                                                                      |
| Compression set [%]               | ≤ 0.50                                                                                      | ≤ 0.50                                                                                      |
| Strain in compression [%]         | ≈ 1.4                                                                                       | ≈ 1.4                                                                                       |
| Shore-A hardness                  | ≈ 67                                                                                        | ≈ 67                                                                                        |

\*The times apply at a room temperature of 23 °C and a normal relative air humidity of 50 %. Higher temperatures shorten, lower temperatures increase these times.

The working time describes the period from the start of mixing until the time by which the tray must be positioned in the patient's mouth.

## Disinfection of the Automix-Dispenser

The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser is also autoclavable.

## Storage and shelf life

- ▶ Store at room temperature (15–25 °C/ 59–77 °F) in a dry place!
- ▶ Do not store in the fridge or freezer!
- ▶ Protect against exposure to direct sunlight!
- ▶ After the application, leave the used mixing tip on the cartridge to cap unused material!
- ▶ Do not use after the expiration date!

## Composition

Polyvinyl siloxanes, silicium-dioxide-based fillers, pigments, additives, platinum catalyst

## Classification

ISO 4823:2000, type 1: Heavy-bodied consistency

## Packaging

### Not available in the United States

| 1 cartridge @ 380 ml paste, 10 MixStar-Tips                                            |            |
|----------------------------------------------------------------------------------------|------------|
|       | REF 909537 |
|  Fast | REF 909638 |

| 5 cartridges @ 380 ml paste, 50 MixStar-Tips                                           |            |
|----------------------------------------------------------------------------------------|------------|
|       | REF 909538 |
|  Fast | REF 909637 |

| 2 cartridges @ 50 ml paste, 4 Automix-Tips                                             |            |
|----------------------------------------------------------------------------------------|------------|
|       | REF 909766 |
|  Fast | REF 909838 |

| 8 cartridges @ 50 ml paste, 16 Automix-Tips                                            |            |
|----------------------------------------------------------------------------------------|------------|
|       | REF 909837 |
|  Fast | REF 909839 |

## Only available in the United States

| 1 cartridge @ 380 ml paste, 10 MixStar-Tips                                                 |            |
|---------------------------------------------------------------------------------------------|------------|
|            | REF 999537 |
|  Quad Fast | REF 999638 |

| 4 cartridges @ 50 ml paste, 12 Automix-Tips                                                 |            |
|---------------------------------------------------------------------------------------------|------------|
|            | REF 999766 |
|  Quad Fast | REF 999838 |

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**DMG** Chemisch-Pharmazeutische Fabrik GmbH