Honigum-Putty

Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	12
Mode d'emploi – FRANÇAIS	20
Istruzioni per l'uso – ITALIANO	29
Instrucciones de uso – ESPAÑOL	37
Instruções de utilização – PORTUGUÊS	46
Gebruiksaanwijzing – NEDERLANDS	55
Brugsanvisning – DANSK	63
Bruksanvisning – SVENSKA	71
Instrukcja użycia – POLSKI	79
Инструкция по применению – РУССКИЙ	88

Produktbeschreibung

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Vinylpolysiloxane, Hydrogenpolysiloxane, Siliciumdioxid, Pigmente, Additive, Platinkatalysator

Klassifikation

ISO 4823:2000, Type 0: Putty consistency

Handelsformen

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1 Kartusche à 380 ml Paste, 10 MixStar-Tips	
Honigum-Putty	REF 919374

5 Kartuschen à 380 ml Paste, 50 MixStar-Tips	
Honigum-Putty	REF 919375

2 Dosen à 450 ml Basis- und Katalysatorpaste	
Honigum-Putty Rigid Fast	REF 919366
Honigum-Putty Soft	REF 919364
Honigum-Putty Soft Fast	REF 919365

8 Dosen à 450 ml Basis- und Katalysatorpaste	
Honigum-Putty Rigid Fast	REF 919371

8 Dosen à 450 ml Basis- und Katalysatorpaste	
Honigum-Putty Soft	REF 919369
Honigum-Putty Soft Fast	REF 919370

1 Flasche à 10 ml Tray Adhesive	REF 909394
---------------------------------	------------

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1 Kartusche à 380 ml Paste, 10 MixStar-Tips	
Honigum-Putty	REF 999374

2 Dosen à 300 ml Basis- und Katalysatorpaste	
Honigum-Putty Rigid Fast	REF 999366
Honigum-Putty Soft Fast	REF 999365

1 Flasche à 10 ml Tray Adhesive	REF 999394
---------------------------------	------------

Product description

Honigum-Putty is an impression material based on addition curing silicones.

The material is available as a hand-mix version and in the ready-to-use pre-filled MixStar cartridge, which is suitable for use in the DMG MixStar-eMotion and other commercially available mixing machines.

Indications

- Crown and bridge impressions
- Inlay and onlay impressions

Basic safety instructions

- For dental use only!
- Keep away from children!


Interactions

- Rinse off residues of retraction solutions thoroughly with water because they can increase the setting time in some instances.


- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Never allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE based gloves.
- Do not mix Honigum-Putty with condensation curing silicone or polyether impression materials.

Using the MixStar cartridge in the DMG MixStar-eMotion

- ▶ Follow the instructions for use for the „MixStar-eMotion“!

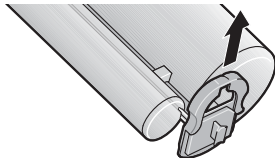
1. Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge.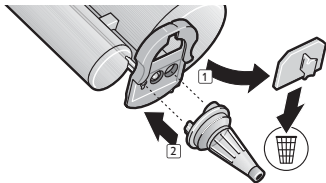


2. Remove the cap from the cartridge and discard. Attach a new mixing tip.

Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.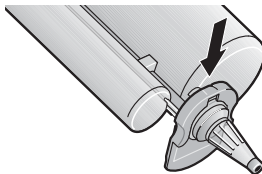

3. Secure the mixing tip by sliding the locking mechanism into the locked position.

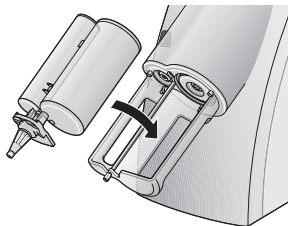4. Insert the cartridge into the MixStar-eMotion.
A brief acoustic signal confirms the correct insertion of the cartridge.

Note: Prepare fresh cartridges for use by extruding and discarding the first 3 cm of impression material.

Note: Leave the used mixing tip on the cartridge after use. It serves as a cap.

Using the MixStar cartridge in mixing units from other manufacturers

- ▶ Follow the instructions of the mixing unit you are using!
- ▶ First insert the cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- ▶ In this case proceed as described in the section “Changing the mixing tip” section on page 14

Changing the mixing tip

- ▶ Follow the instructions of the mixing unit you are using!

Note: Attach a new mixing tip before each mixing process.

1. Slide the locking mechanism upwards.
2. Remove the old mixing tip and any hardened material blocking the cartridge openings.
3. Place the mixing tip securely on the respective cartridge opening.

Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

4. Lock the mixing tip in place with the locking mechanism.

Using the hand-mix system

1. Take equal amounts of the base paste and the catalyst paste out of the container, always

- using the colored spoon for the base paste and the white spoon for the catalyst paste.
2. **Mixing:** At room temperature, knead the base and catalyst pastes in a ratio of 1:1 with the finger tips for 30 seconds, until the color is homogenous.
- ▶ Reseal the container immediately after use! Do not mix up the container lids!

Instructions for use

- ▶ For the individual times refer to the “Technical data” section on page 17.
- ▶ Honigum-Putty can be used in all common impression trays.
- ▶ In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive for A-silicon (e.g. Tray Adhesive, DMG) should be used.

Recommended use

Double-mix impressions

1. Fill the tray with Honigum-Putty.

2. Syringe the preparation with Honigum-Light. Observe the associated instructions for use!

Note: The filling of the tray and syringing should be completed at the same time as the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

3. Position the filled tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.
4. Remove the impression from the mouth once the setting time has elapsed, refer to “Removal” section on page 16.

Corrective impressions

Note: No more than 30 minutes are permitted to pass between the pre-impression and correction impression, otherwise it is not possible to guarantee a firm bond between the two impression materials. Cleaning with alcohol is also not permitted between pre-impression and correction impression.

1. Cut generous drainage channels into the pre-impression and remove imperfections.
2. After experimental repositioning, thoroughly rinse the impression with water and dry.
3. Apply Honigum-Light around the preparation and on the pre-impression. Observe the associated instructions for use!
4. Position the filled tray in the mouth within the working time, slowly and using very little force, and exert brief initial pressure.
5. Remove the impression from the mouth once the setting time has ended, refer to “Removal” section on page 16.

Removal

1. Remove the impression parallel to the longitudinal axis of the prepared teeth.
2. Then proceed as described under “Cleaning and disinfecting the final impression” section on page 16.

Cleaning and disinfecting the final impression

- ▶ Only disinfect the final impression with dipping solutions.
DMG recommends the use of a 2% glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression material.
- ▶ Follow the instructions of the disinfectant solution’s manufacturer precisely!
- ▶ Leave the final impression to dry and store until casting, refer to “Storage of the impression” section on page 17.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

- The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873, Type 3; e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873, Type 4; e.g. Fujirock II, GC). The impression can be copper or silver-plated.

Storage of the impression

- Store the impression in a dry place at room temperature (15–25 °C/ 59–77 °F)!
- Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

Technical data

Honigum-Putty				
		Soft	Soft Fast	Rigid Fast
Mixing time [s]	N. A.	≈ 30	≈ 30	≈ 30
Working time* [min]	≤ 1:45	≤ 3:15	≤ 1:45	≤ 1:45
Time in the mouth [min]	≥ 3:15	≥ 3:15	≥ 1:45	≥ 1:45
Dimensional change after 24h [%]	≤ 0.5	≤ 0.5	≤ 0.5	≤ 0.5
Compression set [%]	≤ 0.6	≤ 0.5	≤ 0.5	≤ 0.5
Strain in compression [%]	≈ 2.0	≈ 2.2	≈ 2.2	≈ 1.7

*The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

The working time describes the period from the start of mixing until the time by which the tray must be positioned in the patient's mouth.

Storage and shelf life

- ▶ Store in a dry place at room temperature (15–25 °C/ 59–77 °F)!
- ▶ Do not store in the refrigerator or freezer!
- ▶ Protect against exposure to direct sunlight!
- ▶ Leave the used mixing tip on the cartridge after use. It serves as a cap.
- ▶ Reseal the jar immediately after use!
- ▶ Do not use after the expiration date!

Composition

Vinyl polysiloxanes, Hydrogen polysiloxanes, silicium dioxide, pigments, additives, platinum catalyst

Classification

ISO 4823:2000, type 0: Putty consistency

Available in the following forms

Not available in the USA

1 cartridge with 380 ml paste, 10 MixStar-Tips

Honigum-Putty	REF 919374
---------------	------------

5 cartridges with 380 ml paste, 50 MixStar-Tips

Honigum-Putty	REF 919375
---------------	------------

2 jars with 450 ml base and catalyst paste

Honigum-Putty Rigid Fast	REF 919366
--------------------------	------------

Honigum-Putty Soft	REF 919364
--------------------	------------

Honigum-Putty Soft Fast	REF 919365
-------------------------	------------

8 jars with 450 ml base and catalyst paste

Honigum-Putty Rigid Fast	REF 919371
--------------------------	------------

Honigum-Putty Soft	REF 919369
--------------------	------------

8 jars with 450 ml base and catalyst paste	
Honigum-Putty Soft Fast	REF 919370

1 bottle with 10 ml Tray Adhesive	REF 909394
-----------------------------------	------------

Only available in the USA

1 cartridge with 380 ml paste, 10 MixStar-Tips	
Honigum-Putty	REF 999374

2 jars with 300 ml base and catalyst paste	
Honigum-Putty Rigid Fast	REF 999366
Honigum-Putty Soft Fast	REF 999365

1 bottle with 10 ml Tray Adhesive	REF 999394
-----------------------------------	------------

which he or she practices to use or order the use of this device.

Caution: US Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in

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DMG Chemisch-Pharmazeutische Fabrik GmbH